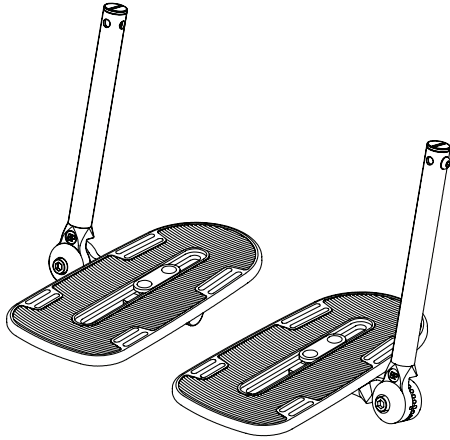
Adjustable-Angle Footplates

INSTALLATION AND USER'S INSTRUCTIONS

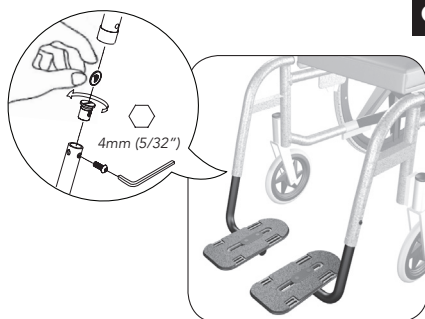
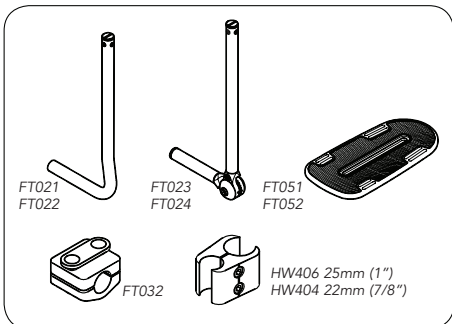
These instructions provide important information for the safe use and maintenance of all Bodypoint Adjustable-Angle Footplate kits and accessories. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products and a list of distributors outside the USA go to www.bodypoint.com.

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PACKAGE CONTENTS

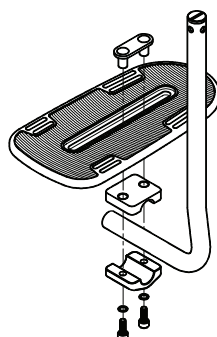
Package may contain one or more of the following:

- Extension Tubes
- Footplates
- Footplate Clamp
- Mounting Clamp Kit

Or a complete Footplate Kit. Art No.: FT101-, FT102-, FT105-, FT106-

INTERNAL MOUNTING

On wheelchairs which accept $\varnothing 19\text{mm}$ ($3/4"$) extension tubes, Adjustable-Angle Footplates and Extension Tubes can be installed internally by inserting them into the wheelchair's legrest hanger tubes. Bodypoint Extension Tubes are fitted with a threaded plug for screw mount installations. Turn the plug to align with the suitable hole in the wheelchair's legrest hanger tube.



EXTERNAL MOUNTING

Adjustable-Angle Footplates and Extension Tubes can be installed externally by using our optional Mounting Clamps. Attach Mounting Clamp to wheelchair legrest hanger tube or directly to front frame of wheelchair and then attach Extension Tube.

FOOTPLATES AND FOOTPLATE CLAMP

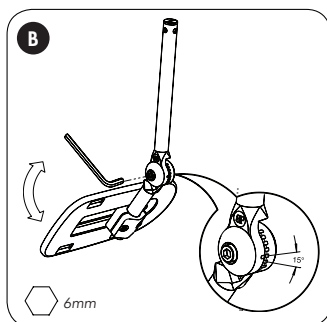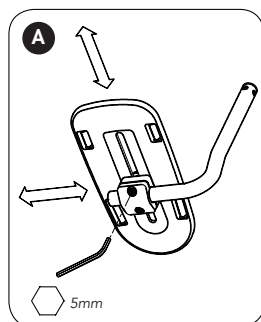Assemble Footplates as shown. With the screws loose, adjust the Footplates as required to fit the user. Tighten screws equally to ensure good clamping.

ADJUSTMENT

- Adjust Footplate position using the Footplate Clamp (All models).
- Adjust angle of extension tube at joint, using the 15° markings for reference. Tighten to 22Nm (195 lbf-in). (Fulcrum models)

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MDSS GmbH